

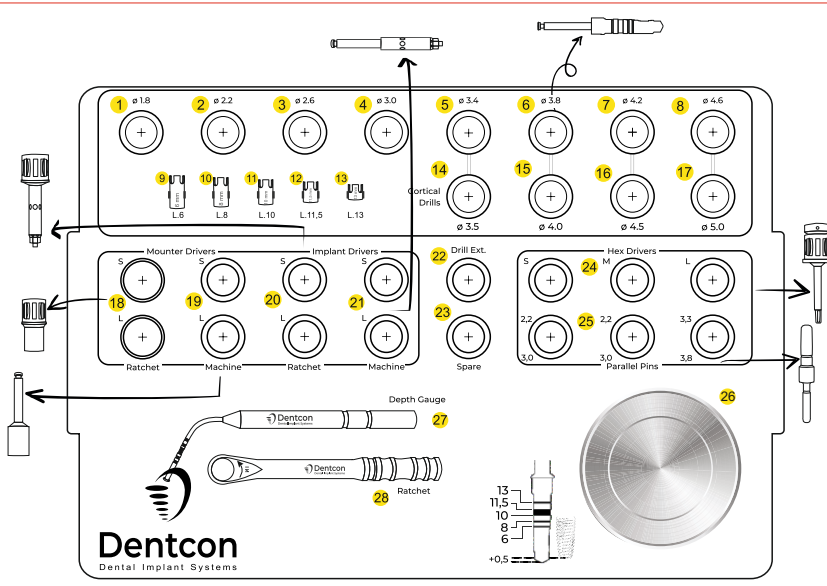


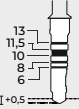

Code Component Name

- | | | |
|--------------------|--------------------------------|--------------------------------|
| 1 Start Drill | 11 Stopper L. 10 mm | 21 Implant Machine Adaptor SJL |
| 2 Drill 02.2 | 12 Stopper L. 11.5 mm | 22 Drill Extender |
| 3 Drill 02.6 | 13 Stopper L. 13 mm | 23 Spare |
| 4 Drill 03.0 | 14 Cortical Drill 03.5 | 24 Hex Drivers |
| 5 Drill 03.4 | 15 Cortical Drill 04.0 | 25 Parallel Pin (Thin - Thick) |
| 6 Drill 03.8 | 16 Cortical Drill 04.5 | 26 Bowl |
| 7 Drill 04.2 | 17 Cortical Drill 05.0 | 27 Depth Gauge |
| 8 Drill 04.6 | 18 Moulder Ratchet Adaptor SJL | 28 Ratchet Wrench |
| 9 Stopper L. 6 mm | 19 Moulder Machine Adaptor SJL | |
| 10 Stopper L. 8 mm | 20 Implant Ratchet Adaptor SJL | |

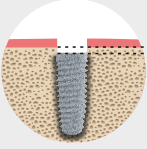

Dentcon Dental İmplant Sistemleri'nin frezleri fiştür boylarından 0,5 mm daha uzundur. Planlama bu şekilde düzenlenmelidir.


Dentcon Dental Implant Systems drills are 0.5 mm longer than fixture lengths. Planning should be made accordingly.



Please scan the QR code for eIFU and other documents


www.dentimplant.com.tr

Ø3.5

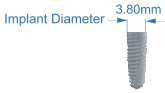

	Start Drill	Ø2.2	Ø2.6	Ø3.0	Ø3.4	Ø3.8	Ø4.2	Ø4.6	Cortical Drill Ø3.5	Fixture Ø3.5
D1	•	•	•	•	•				•	
D2	•	•	•	•	•				•	
D3	•	•	•	•	•				•	
D4	•	•	•	•	•				•	

Ø4.0

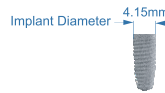


	Start Drill	Ø2.2	Ø2.6	Ø3.0	Ø3.4	Ø3.8	Ø4.2	Ø4.6	Cortical Drill Ø4.0	Fixture Ø4.0
D1	•	•	•	•	•	•			•	
D2	•	•	•	•	•	•			•	
D3	•	•	•	•	•	•			•	
D4	•	•	•	•	•	•			•	

Ø4.5



	Start Drill	Ø2.2	Ø2.6	Ø3.0	Ø3.4	Ø3.8	Ø4.2	Ø4.6	Cortical Drill Ø4.5	Fixture Ø4.5
D1	•	•	•	•	•	•	•		•	
D2	•	•	•	•	•	•	•		•	
D3	•	•	•	•	•	•	•		•	
D4	•	•	•	•	•	•	•		•	

Ø5.0

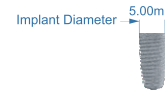

	Start Drill	Ø2.2	Ø2.6	Ø3.0	Ø3.4	Ø3.8	Ø4.2	Ø4.6	Cortical Drill Ø5.0	Fixture Ø5.0
D1	•	•	•	•	•	•	•	•	•	
D2	•	•	•	•	•	•	•	•	•	
D3	•	•	•	•	•	•	•	•	•	
D4	•	•	•	•	•	•	•	•	•	